



TECHNICAL DATA SHEET

BMMSM1200 SERIES
DISPOSABLE SURGICAL FACE MASK

BS EN 14683:2019+AC TYPE II
MEDICAL DEVICE-CLASS I



CONTACT INFORMATION

Global Health and Safety Ltd.
Kenrich Business Park, Unit 2&3,
Elizabeth Way, Harlow, CM19 5TL,
United Kingdom

Tel: + 44 020 3545 0700


www.globalhealthandsafety.co.uk
info@globalhealthandsafety.co.uk

I. PRODUCT OVERVIEW

British Mask BMMSM 1200 series three layers medical grade disposable surgical masks engineered for exceptional comfort and maximum protection, using only highest quality technical non woven fabrics.

Each mask offers $\geq 98\%$ bacterial filtration efficiency and complies with the BS EN 14683:2019+AC Type II standard.

Global Health & Safety surgical masks offer proven filtration and may help to minimize the risk of transmission of viruses without compromising comfort and breathability.

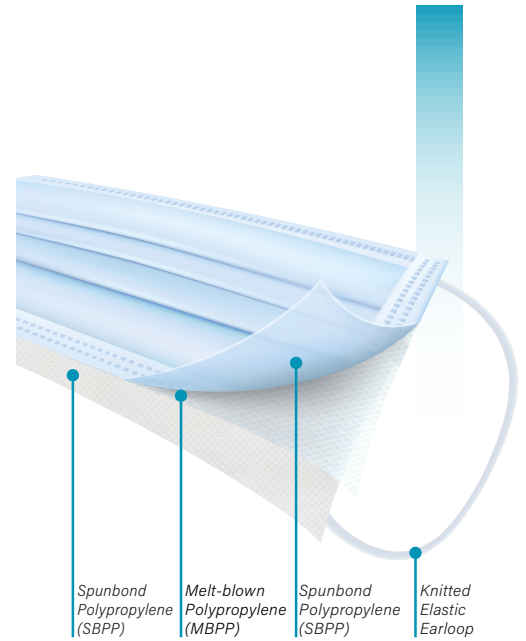

II. INTENDED USE

Medical face masks are used to limit the transmission of infective agents from staff to patients during procedures and other medical settings. Medical face masks may also be intended to be worn by patients and other persons to reduce risk of spread of infections, particularly in epidemic or pandemic situations.

Global Health and Safety Surgical Masks are intended to be used as short term, non-invasive, non-sterile protective medical device. BS EN 14683:2019 +AC:2019 masks are intended to be used in setting where exposure to fluid is not a risk.

III. PRODUCT SPECIFICATION

Surgical Mask BS EN 14683:2019+AC TYPE II				
CHARACTERISTIC		SPECIFICATION		
DESCRIPTION		Rectangular pleat style face masks with a shapeable nose-piece and two earloops present, one on each side, in order to secure mask in place.		
MATERIAL	Mask Body Composition	OUTER MATERIAL	Spun-bond Polypropylene (SBPP)	
		FILTER MATERIAL	Melt-blown Polypropylene (MBPP)	
		INNER MATERIAL	Spun-bond Polypropylene (SBPP)	
	Nose-Clip Composition	Aluminium / Plastic Coated		
Elastic Band Composition		Knitted Yarn (Polyamide 75 % Elastane 25 %)		
COLOUR	ITEM CODE		OUTER COLOUR	
	BMMSM101200		WHITE	
	BMMSM101204		PINK	
	BMMSM101205		BLUE	
	BMMSM101206		GREEN	
DIMENSION	Body Size	Length	SPECIFICATIONS	
		Width	175 mm (+/- 2 mm)	
	Length of Earloop		95 mm (+/- 2 mm)	
	Length of Nose - Piece		200 mm	
			100 mm	
Bacterial Filtration Efficiency (BFE)		BFE @ 3,0 μ m large Bacteria $\geq 98\%$		
Differential Pressure (ΔP)		$(\Delta P) < 40$ Pa/cm ²		
Microbial Cleanliness		≤ 30 cfu/g		
PACKAGING	ITEM CODE		MASKS / BOX	BOXES / CASE
	BMMSM1200 SERIES		50 PCS	12 BOXES
				MASKS / CASE
				600 PCS

III. PACKAGING OVERVIEW

Our packaging contains crucial and key information to ensure quality and traceability of each product produced. We use GS1 barcoding which includes GTIN13, GTIN14 and GLN to reduce handling and processing time, ensuring all deliveries reach their correct destination on time.



DIMENSIONS	DEPTH (D)	WIDTH (W)	HEIGHT (H)	CONTENT
BOX	100 mm	190 mm	100 mm	50 pcs Mask
CASE	315 mm	380 mm	200 mm	12 Boxes

IV. IMPORTANT INFO

This medical face mask cannot stop the transmission of all germs in the air, does not eliminate the risk of disease or illness, and does not replace protective measures. Practice good hygiene. Wash your hands frequently and refrain from touching your nose, mouth or eyes. Do not clean or sterilize for re-use.

V. INSTRUCTION FOR USE

- Sanitize or wash your hands with soap and water before taking the mask out of its packaging and putting it on.
- Place a loop around each ear.
- Place the mask carefully. Avoid touching the inner layer.
- Pinch the nose clip to the shape of your nose.
- Pull the bottom of the mask over your chin.
- Ensure the mask covers the bridge of your nose, mouth, and chin snugly but comfortably.
- While in use avoid touching your face or the mask.
- Remove the mask by the ear loops. Do not touch the front of the mask.
- Discard immediately upon removal.
- Sanitize or wash your hands with soap and water immediately after.

VI. WARNINGS

- Inspect the mask before use for any signs of tears or deterioration.
- Do not use damaged mask.
- Single use only. Must be discarded after each use.
- Change the mask as soon as they become damp/humid or if it gets dirty.
- Do not remove the mask to speak.
- Do not use in the presence of a high heat source.
- Do not use except as indicated.
- To be used in a settings where exposure to fluid is not a risk.

VII. REGULATORY

BS EN 14683 : 2019 + AC
MEDICAL DEVICE CLASS I



VIII. STORAGE CONDITIONS

Store at 20°C - 35°C under dry conditions.

Shelf life 5 years.

Global *health*  *safety*[®]

UK MANUFACTURERS OF HEALTH CARE PRODUCTS

 **BRITISH MASK**[®]  **Face Guard**[®] **Flexy**[®]

CONTACT INFORMATION

Global Health and Safety Ltd.
Kenrich Business Park, Unit 2&3,
Elizabeth Way, Harlow, CM19 5TL,
United Kingdom

Tel: + 44 020 3545 0700

www.globalhealthandsafety.co.uk
info@globalhealthandsafety.co.uk