



TECHNICAL DATA SHEET

BMMSM1300 SERIES
DISPOSABLE SURGICAL FACE MASK

BS EN 14683:2019+AC TYPE IIR
MEDICAL DEVICE-CLASS I



CONTACT INFORMATION

Global Health and Safety Ltd.
Kenrich Business Park, Unit 2&3,
Elizabeth Way, Harlow, CM19 5TL,
United Kingdom

Tel: + 44 020 3545 0700


www.globalhealthandsafety.co.uk
info@globalhealthandsafety.co.uk

I. PRODUCT OVERVIEW

British Mask BMMSM 1300 series three layers medical grade disposable surgical masks engineered for exceptional comfort and maximum protection, using only highest quality technical non woven fabrics.

Each mask offers $\geq 98\%$ bacterial filtration efficiency, fluid resistant and complies with the BS EN 14683:2019+AC Type IIR standard.

Global Health & Safety surgical masks offer proven filtration and may help to minimize the risk of transmission of viruses without compromising comfort and breathability.

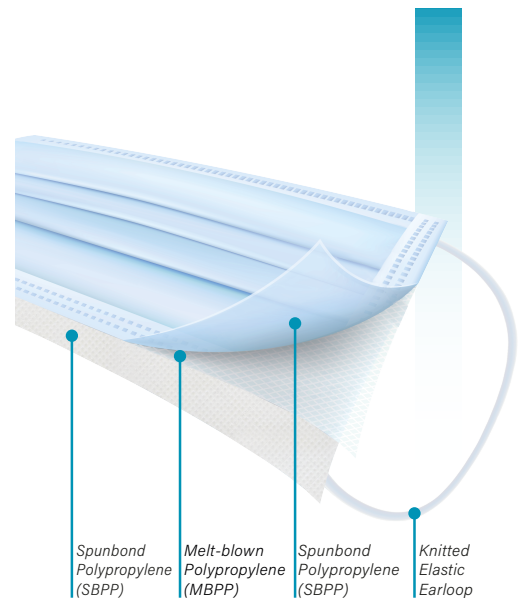

II. INTENDED USE

Medical face masks are used to limit the transmission of infective agents from staff to patients during procedures and other medical settings. Medical face masks may also be intended to be worn by patients and other persons to reduce risk of spread of infections, particularly in epidemic or pandemic situations. Global Health and Safety Surgical Masks are intended to be used as short term, non-invasive, non-sterile protective medical device.

III. PRODUCT SPECIFICATION

Surgical Mask BS EN 14683:2019+AC TYPE IIR					
CHARACTERISTIC		SPECIFICATION			
DESCRIPTION		Rectangular pleat style face masks with a shapeable nose-piece and two earloops present, one on each side, in order to secure mask in place.			
MATERIAL	Mask Body Composition	OUTER MATERIAL	Spun-bond Polypropylene (SBPP)		
		FILTER MATERIAL	Melt-blown Polypropylene (MBPP)		
		INNER MATERIAL	Spun-bond Polypropylene (SBPP)		
	Nose-Clip Composition	Aluminium / Plastic Coated			
Elastic Band Composition		Knitted Yarn (Polyamide 75 % Elastane 25 %)			
COLOUR		ITEM CODE	OUTER COLOUR		
		BMMSM101300	WHITE		
		BMMSM101304	PINK		
		BMMSM101305	BLUE		
		BMMSM101306	GREEN		
DIMENSION	Body Size	Length	175 mm (+/- 2 mm)		
		Width	95 mm (+/- 2 mm)		
	Length of Earloop		200 mm		
	Length of Nose - Piece		100 mm		
	SPECIFICATIONS				
Bacterial Filtration Efficiency (BFE)		BFE @ 3,0 μ m large Bacteria $\geq 98\%$			
Differential Pressure (ΔP)		$(\Delta P) < 60 \text{ Pa/cm}^2$			
Microbial Cleanliness		$\leq 30 \text{ cfu/g}$			
Fluid Resistance		Synthetic Blood Resistant @ 120mmHg ($\geq 16.0\text{kPa}$)			
PACKAGING		ITEM CODE	MASKS / BOX	BOXES / CASE	MASKS / CASE
		BMMSM1300 SERIES	50 PCS	12 BOXES	600 PCS

IV. PACKAGING OVERVIEW

Our packaging contains crucial and key information to ensure quality and traceability of each product produced. We use GS1 barcoding which includes GTIN13, GTIN14 and GLN to reduce handling and processing time, ensuring all deliveries reach their correct destination on time.



DIMENSIONS	DEPTH (D)	WIDTH (W)	HEIGHT (H)	CONTENT
BOX	100 mm	190 mm	100 mm	50 pcs Mask
CASE	315 mm	380 mm	200 mm	12 Boxes

IV. IMPORTANT INFO

This medical face mask cannot stop the transmission of all germs in the air, does not eliminate the risk of disease or illness, and does not replace protective measures. Practice good hygiene. Wash your hands frequently and refrain from touching your nose, mouth or eyes. Do not clean or sterilize for re-use.

V. INSTRUCTION FOR USE

- Sanitize or wash your hands with soap and water before taking the mask out of its packaging and putting it on.
- Place a loop around each ear.
- Place the mask carefully. Avoid touching the inner layer.
- Pinch the nose clip to the shape of your nose.
- Pull the bottom of the mask over your chin.
- Ensure the mask covers the bridge of your nose, mouth, and chin snugly but comfortably.
- While in use avoid touching your face or the mask.
- Remove the mask by the ear loops. Do not touch the front of the mask.
- Discard immediately upon removal.
- Sanitize or wash your hands with soap and water immediately after.

VI. WARNINGS

- Inspect the mask before use for any signs of tears or deterioration.
- Do not use damaged mask.
- Single use only. Must be discarded after each use.
- Change the mask immediately if exposed to blood or any harmful fluid.
- Change the mask as soon as they become damp/humid or if it gets dirty.
- Do not remove the mask to speak.
- Do not use in the presence of a high heat source.
- Do not use except as indicated.

VII. REGULATORY

BS EN 14683 : 2019 + AC
MEDICAL DEVICE CLASS I



VIII. STORAGE CONDITIONS

Store at 20°C - 35°C under dry conditions.
Shelf life 5 years.

Global *health*  *safety*[®]

UK MANUFACTURERS OF HEALTH CARE PRODUCTS

 **BRITISH MASK**[®]  **Face Guard**[®] **Flexy**[®]

CONTACT INFORMATION

Global Health and Safety Ltd.
Kenrich Business Park, Unit 2&3,
Elizabeth Way, Harlow, CM19 5TL,
United Kingdom

Tel: + 44 020 3545 0700

Please consider the environment before printing this document.

GHS_TDS_BMMSM101300_DEC21_EN

www.globalhealthandsafety.co.uk
info@globalhealthandsafety.co.uk