

Face Guard[®]

PRODUCT CARD

FGMSM1300 SERIES
DISPOSABLE SURGICAL FACE MASK

**BS EN 14683:2019+AC TYPE IIR
MEDICAL DEVICE-CLASS I**

 ENGLISH

CONTACT INFORMATION

Global Health and Safety Ltd.
Kenrich Business Park, Unit 2&3,
Elizabeth Way, Harlow, CM19 5TL,
United Kingdom

Tel: + 44 020 3545 0700

www.globalhealthandsafety.co.uk
info@globalhealthandsafety.co.uk

I. PRODUCT OVERVIEW

Surgical Mask BS EN 14683:2019+AC TYPE IIR

Face guard FGMSM 1300 series three layers medical grade disposable surgical masks engineered for exceptional comfort and maximum protection, using only highest quality technical non woven fabrics.

Each mask offers 98% bacterial filtration efficiency, fluid resistant and complies with the BS EN 14683:2019+AC Type IIR standard.

Global Health & Safety surgical masks offer proven filtration and may help to minimize the risk of transmission of viruses without compromising comfort and breathability.



II. PRODUCT SPECIFICATION

Surgical Mask BS EN 14683:2019+AC TYPE IIR				
CHARACTERISTIC		SPECIFICATION		
DESCRIPTION		Rectangular pleat style face masks with a shapeable nose-piece and two earloops present, one on each side, in order to secure mask in place.		
MATERIAL	Mask Body Composition	OUTER MATERIAL	Spun-bond Polypropylene (SBPP)	
		FILTER MATERIAL	Melt-blown Polypropylene (MBPP)	
		INNER MATERIAL	Spun-bond Polypropylene (SBPP)	
	Nose-Clip Composition	Aluminium / Plastic Coated		
Elastic Band Composition		Knitted Yarn (Polyamide 75 % Elastane 25 %)		
COLOUR	ITEM CODE		OUTER COLOUR	
	FGMSM101300		WHITE	
	FGMSM101304		PINK	
	FGMSM101305		BLUE	
	FGMSM101306		GREEN	
DIMENSION	Body Size	Length	ITEM DIMENSIONS	
		Width	OTHER ITEMS	
	Length of Earloop		200 mm	
	Length of Nose - Piece		100 mm	
	Length		175 mm (+/- 2 mm)	
Bacterial Filtration Efficiency (BFE)		BFE @ 3,0 µm large Bacteria ≥98%		
Differential Pressure (ΔP)		(ΔP) <60 Pa/cm2		
Microbial Cleanliness		≤30 cfu/g		
Fluid Resistance		Synthetic Blood Resistant @ 120mmHg		
Maximum Wearing Time		4 Hours		
PACKAGING	ITEM CODE		MASKS / BOX	BOXES / CASE
	FGMSM1300 SERIES		50 PCS	12 BOXES
				MASKS / CASE
				600 PCS

III. PACKAGING OVERVIEW

All our products are constructed with the highest standards of protection and breathability using only raw materials from reputable manufacturers with top of the line quality. Advanced technological solutions are at the core of our company operations. Our innovative packaging contains crucial and key information to ensure quality and traceability of each product produced. We use GS1 barcoding which includes GTIN13, GTIN14 and GLN to reduce handling and processing time, ensuring all deliveries reach their correct destination on time.

EACH BOX CONTAINS;		FEATURES
LOT NO	✓	One Size
EXPIRY DATE	✓	Soft Touch
PRODUCTION DATE	✓	Latex-Free
MANUFACTURER INFO	✓	Disposable
GTIN 13 BARCODING (BOX)	✓	Three Layers
GTIN 14 BARCODING (CASE)	✓	High Breathability
TEMPER EVIDENT SECURITY SEAL	✓	Comfortable Earloops
SECURITY HOLOGRAM	✓	Hi-Tech Premium Melt Blown Filter Media



A TOP VIEW

B FRONT VIEW

C RIGHT SIDE VIEW

D BACK VIEW

E LEFT SIDE VIEW

A TOP VIEW

B FRONT VIEW

C RIGHT SIDE VIEW

D BACK VIEW

E LEFT SIDE VIEW

DIMENSIONS	DEPTH (D)	WITH (W)	HEIGHT (H)	CONTENT
BOX	100 mm	190 mm	100 mm	50 pcs Mask
CASE	315 mm	380 mm	200 mm	12 Boxes

IV. IMPORTANT INFO

This medical face mask cannot stop the transmission of all germs in the air, does not eliminate the risk of disease or illness, and does not replace protective measures. Practice good hygiene. Wash your hands frequently and refrain from touching your nose, mouth or eyes. Do not clean or sterilize for re-use.

Each mask is single-use only and should be disposed immediately after use. You should also remove and replace your mask when they become moist, as this can impact the effectiveness of the filter.

V. DIRECTIONS FOR USE

- Wash your hands with soap and water, then take the mask out of its packaging.
- Hold the mask by ear loops. Place a loop around each ear.
- Avoid touching the inner layer while putting the mask on.
- Pinch the nose clip to the shape of your nose.
- Pull the bottom of the mask over your chin.
- Mask should cover your mouth, nose and chin snugly but comfortably.

VI. HOW TO REMOVE THE FACE MASK

- Clean your hands with soap and water or hand sanitiser before touching the mask.
- Avoid touching the front of the mask. Only touch the ear loops.
- Hold both of the ear loops and gently lift and remove the mask from your face.
- Dispose of the mask in the bin.
- Clean your hands with soap and water or hand sanitiser.

VII. WARNINGS

- Single use only. Must be discarded after each use.
- Change the mask immediately if exposed to blood or any harmful fluid.
- Do not use except as indicated.
- Do not use in the presence of a high heat source.
- Check the mask before use for any signs of damage or deterioration.
- Store in a dry and cool place.
- Non-sterile.

VIII. REGULATORY

BS EN 14683 : 2019 + AC
MEDICAL DEVICE CLASS I



IX. STORAGE CONDITIONS

Store under ambient temperature and dry conditions. Avoid excessive heat.

Shelf life 2 years after the date of production.

Global *health*  *safety*

Maximum **Comfort** and Highest **Protection**

CONTACT INFORMATION

Global Health and Safety Ltd.
Kenrich Business Park, Unit 2&3,
Elizabeth Way, Harlow, CM19 5TL,
United Kingdom

Tel: + 44 020 3545 0700

www.globalhealthandsafety.co.uk
info@globalhealthandsafety.co.uk